**PX PREMIUM YELLOW**

CE 1250

**PX DENTAL SA**
CH-2300 LA CHAUX-DE-FONDS

JAUNE	ALLIAGE DENTAIRE CÉRAMO MÉTALLIQUE
GELB	DENTAL AUFBRENNLEGIERUNGEN
GIALLO	LEGA DENTALE PER CERAMICA
YELLOW	DENTAL BONDING ALLOY
AMARILLO	ALEACIÓN DENTAL CERAMO-MÉTALICA

% = Au: 76.5, Pt: 13.5, Ag: 6.5, Zn: 2.5, Ta, Ir, Ru: <1%**17.8 g/cm³****1020-1120° C****14.4/14.8 µm/m° C****ISO 9693**Fabriqué par / Hergestellt von / Fabricato da /
Made by / Fabricado por**PX DENTAL SA**

Rue Numa-Droz 150, 2300 La Chaux-de-Fonds

T. +41 (0)32 924 21 20 F +41 (0)32 924 21 29

www.px dental.com

PX DENTAL SA
CH-2300 LA CHAUX-DE-FONDS

Mode d'emploi

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1. Waxing up, spruing, investing

- Follow the general prescriptions for bonding alloys.
- The ultimate thickness of the cast coping should not be less than 0.3 mm.
- Use only a high-heat phosphate-bonded investment material. For the alloy PX PREMIUM YELLOW we recommend to use the PX Extravest investment.

2. Preheating

- Observe manufacturer's recommendations with regard to setting times, temperature levels, etc.
- Keep the cylinder at 750-800° C for a soaking period of 30-45 min.

3. Crucibles and casting buttons

- Use a ceramic crucible.
- Add at least 1/3 of new alloy. When remelted casting buttons should be perfectly clean.

4. Melting

- For PX PREMIUM YELLOW casting temperature is 1320° C.
- The use of crystallized boric acid is recommended.
- Bench cool the casting ring.

5. Treatment of the casting

- Devesting: by blasting (grain size of 50-125 µm).
- Following casting or soldering, the frame may be pickled.
- Preparing the framework and cleaning: trim the framework preferably using carbide burs or fine carborundum stones at low speed another blasting (grain size of 50-125 µm) and preceding oxidation clean the frameworks in an ultrasonic unit or with a steam cleaner.

6. Oxidation

- 10 min under vacuum at 900° C.

7. Porcelain application

- Porcelain applications are subject to the manufacturers instructions.
- PX PREMIUM YELLOW is compatible with most known porcelain brands (high fusing).

8. Soldering

- Pre-bond: use PX SOLD LF 950 solder at 950° C with a flux.
- Post-bond: use PX SOLD 740 solder at 790° C with a flux.

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